

# COVID-19 ANTIGEN TESTS

- The COVID-19 Antigen Rapid Test Device detects the presence of SARS-CoV-2 viral nucleoprotein antigens in nasal and nasopharyngeal secretions.)
- In just 15 minutes, individuals can determine a positive or negative result. If positive, individuals should adhere to their local public health guidelines and verify results with a PCR test.)
- Antigens are generally detectable in nasal secretions during the acute phase of infection. Positive results indicate the presence of viral antigens, but other diagnostic information is necessary to determine infection status.)
- Negative results should be treated as presumptive, and do not rule out SARS-CoV-2 infection and should not be used as the sole basis for treatment or patient management decisions, including infection control decisions.)
- All Rapid Tests are FINAL SALE – we cannot offer cancellations or refunds.)
- New pricing in effect as of February 9, 2022


**IN STOCK!**



**Canadian Made Artron COVID-19  
Antigen Test (25 tests per box)**




## HOW TO PERFORM THE TEST

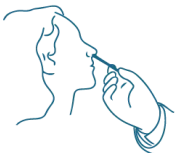


### COLLECT SAMPLE

Insert swab in nostril (2cm) and move in circular motions for 15 seconds. Repeat in other nostril.

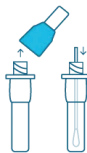

### INSERT SAMPLE

Remove the cap from the vial and place the swab in the vial for 2 minutes.

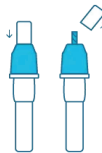

### REMOVE SAMPLE

Remove the swab and discard in the included biohazard bag. Replace the cap on the vial and remove the dropper cap.



### DROP

Squeeze out two droplets out of the vial onto the designated area of the cassette. Wait 15 minutes before reading results.

**Usage: Warehouse, Outdoors, Office, Events, Conferences, Work, Highly Populated Areas.**

Production Time: 1 Working Day • Please use Promo code: 222JUL • Offer Expires July 31st 2022